



1. Place glass slides in plastic slide containers and **tape** shut. Please **do not** use cardboard slide containers—slides will break in transit.

Alternatively, if shipping fluid, please submit in EDTA vacutainer tubes placed inside straphone mailers or other cushioning, inside a ziplock-type bag adjacent to a cold pack.

2. Place the online printed or handwritten request and cytology specimen into the padded EVP slide shipper envelope.
3. Place the slide shipper envelope into the FedEx clinical pak envelope with 2-day or overnight shipping label.

If you have questions, call us at **865-201-6460**.

